

TT380 IUD Cannula Update


Update on the New Style Insertion Cannula


There have been some minor changes to the TT380 Copper IUD insertion Cannula. These changes will assist you, as the Clinician, to more easily obtain correct placement of the IUD under most conditions.

THESE CHANGES DO NOT ALTER THE INSERTION TECHNIQUE FOR THE IUD

The new insertion cannula is slightly longer than the original allowing for more flexibility when inserting & the handle is slightly shorter. I.e., the overall length is the same.


Picture

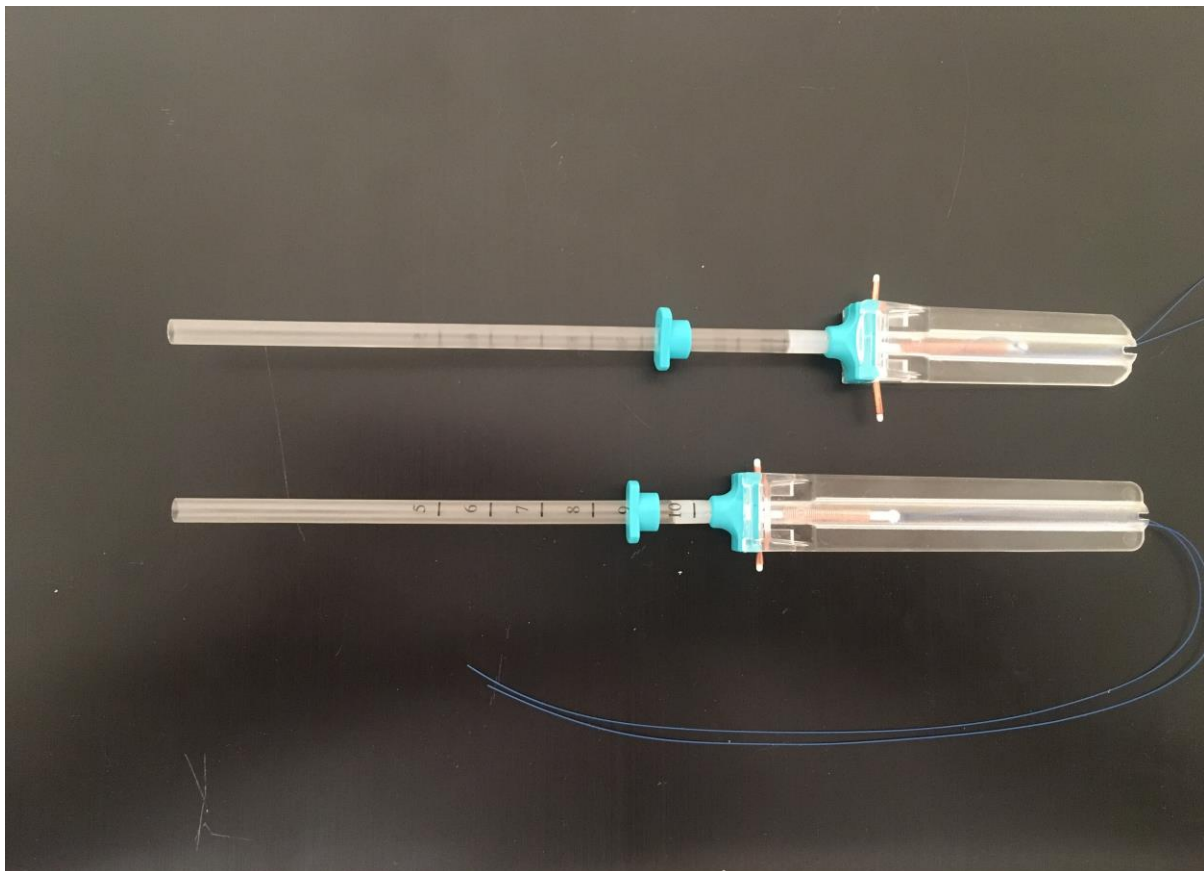

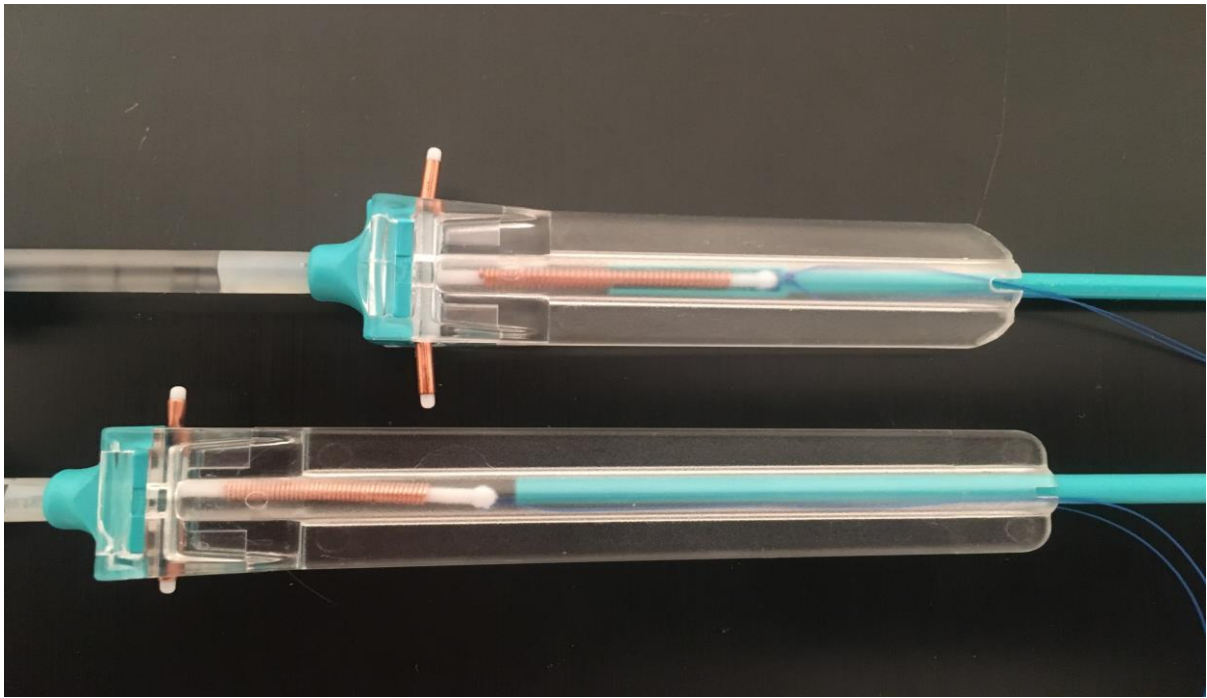

The Plunger now has a scoop at the end to hold the body of the IUD. This will assist in the placement of the IUD and reduce the possibility of the insertion cannula bending at the IUD and plunger junction, at insertion.

Picture

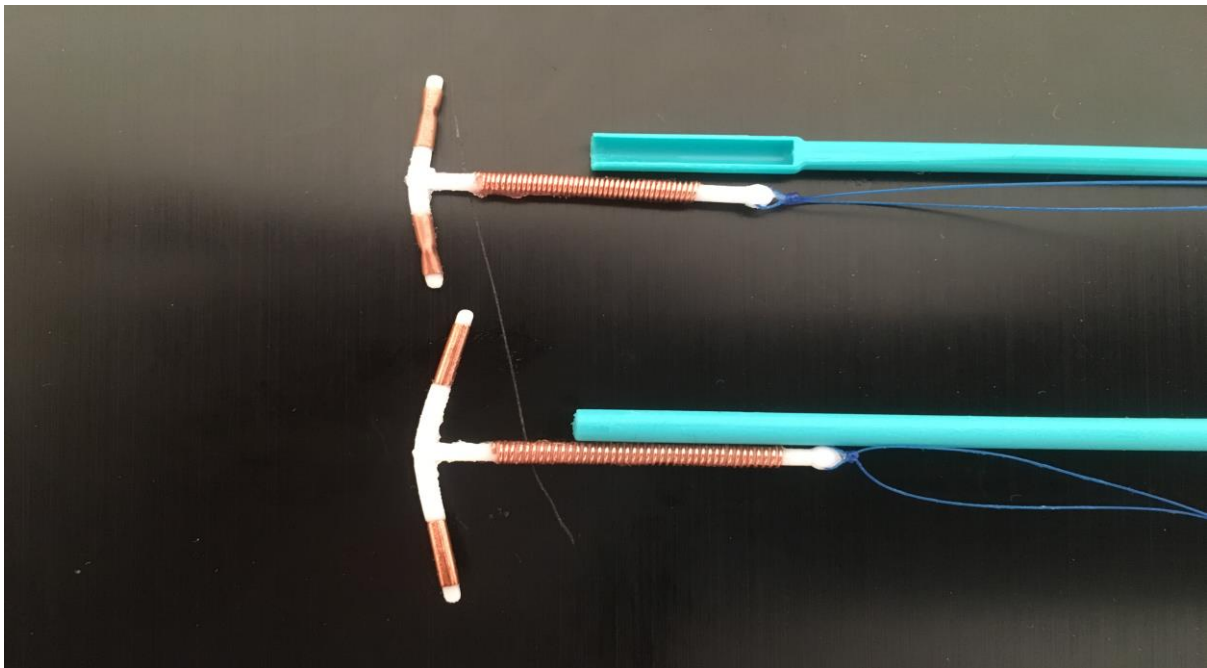

These changes only affect the TT380 Standard & Short the LOAD375 remain the same.

Medical Industries Australia
www.medind.com.au
sales@medind.com.au